

Front of
Instructions

6.5 in

Mountain fold

Valley fold

Mountain fold

Valley fold

Mountain fold

Trim

Bleed (0.125 in)

Back of
Instructions





DATE	03/09/21
CLIENT	Qvin
PROJECT	Instructions - 6-fold accordian
SIZE	22.5 in x 6.5 in (flat) 3.75 in x 6.5 in (folded)
COLOR	CMYK

MECHANICAL	CROP/TRIM
	BLEED
	FOLD (Mountain)
	FOLD (Valley)

Download the Qvin app and register your Q-Pad kit.

1


- Start by downloading the Qvin app.
- Register your Q-Pad kit with the ID on Q-Pad 1 before use.
- If you don't have access to a smartphone, please fill out the **Identification Form**, which can be found in the Return Pouch.

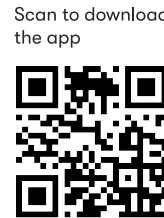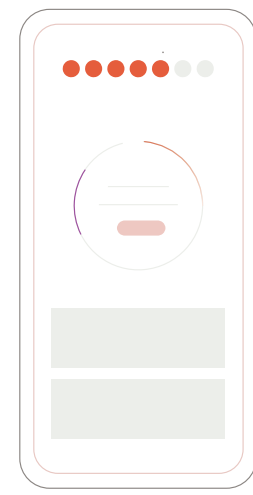

Use your Q-Pads on a heavy flow day.

2


- Before you use your Q-Pads, open the Qvin App and push "Start using".
- Start with Q-Pad 1 and wear it until the oval on the pad is covered.
- Pull the tab to remove the Q-Strip.
- Place the Q-Strip in the provided Q-Strip Container immediately.
- Use Q-Pad 2 and repeat this process.
- Don't worry about the amount of sample on the Q-Strips. Send back whatever you have.

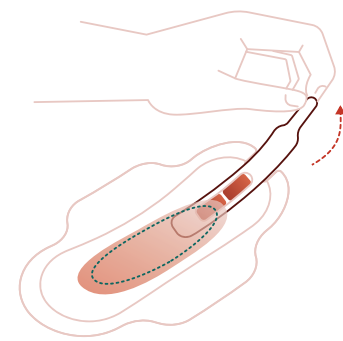

Mail your Q-Strips within 72 hours.

3

- Put the Q-Strip Container inside the prepaid Return Pouch.
- Peel off the liner and seal the Return Pouch.
- Drop it off at the Shipping Provider within 72 hours.

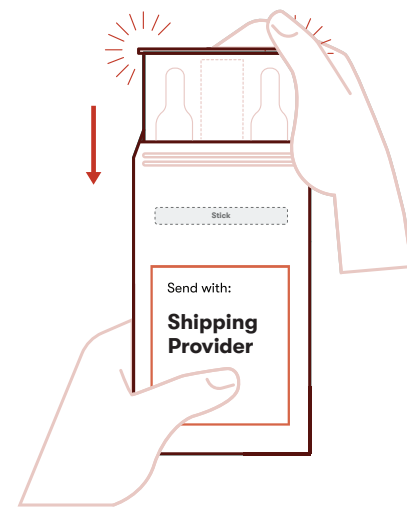

Qvin

3.75 in

3.75 in

3.75 in

3.75 in

3.75 in

3.75 in

22.5 in

Warnings and Limitations

Collection Device

This device collects naturally occurring vaginal and menstrual fluids across a wide range of bleeding or discharge patterns.

Keep Q-Strip Container in pouch

Use only the designated Return Pouch and Q-Strip Container to ship the Q-Strips. Keep the Q-Strip Container in the pouch at all times.

Excessive urine

If you experienced excessive urination that has transferred to the Q-Pad or have urinated on the Q-Pad, consult customer service before submitting your Q-Pad to the laboratory.

Wear the Q-Pad for max 8h

As with any other feminine hygiene product, it is not advised to wear your Q-Pad for longer than 8 hours.

Ship back within 72 hours

Please ship your Q-Strips back within 72 hours of starting Q-Pad 1.

Storage and Stability

Components in the Q-Pad Kit are stable until the expiration date printed on the packaging when stored at room temperature (15-30°C, 30-70% RH). Open kits are usable for up to 60 days when stored at room temperature (15-30°C, 30-70% RH).

Engage your healthcare provider

The Q-Pad provides valuable insights but should never be the sole basis for medical decisions. Do not change medications, treatments, or health routines without consulting a qualified healthcare professional. Always discuss your results with your doctor to determine the best course of action for your health.

Avoid products on your vaginal area while using the Q-Pad

Use of any products such as creams, lubricants, moisturizers, wipes, or deodorants, in or on your vagina before or during the use of the Q-Pad may cause inaccurate results. Ensure your intimate area is clean before using the Q-Pad.

Single-Use Disposal of the Q-Pad

The Q-Pad is intended for single use only. After removing the Q-Strip, wrap the used Q-Pad and dispose of it in accordance with local regulations for menstrual pads. Do not flush the Q-Pad. Reuse may increase the risk of infection.

Irritation

If irritation occurs, discontinue use. If symptoms persist, consult your healthcare provider.

In the USA, for In-Vitro Diagnostic Use Only

Need help?

For most common questions, please visit support.qvin.com

Medical Emergency and Medical Device Issues

Contact your healthcare provider immediately. Any serious incident that has occurred in relation to this device should be reported to the manufacturer and to the competent authority of the country where the user and/or patient is located.

In the EU, report to the national competent authority. In the UK, report to the MHRA. In the USA, report to the FDA.

Still need help

Email us anytime at care@qvin.com.

Prefer to talk

Call +1-833-FOR-QVIN (1-833-367-7846) Monday-Friday, 9 AM-5 PM PT.

Symbols Glossary

EC REP

Authorized representative in the ECRU

UK REP

Authorized representative in the United Kingdom



Biological risks



Caution



CE Mark



Consult Instructions for Use



Contains biological material of human origin



Date of manufacturer



Do not re-use



Importer



In-Vitro Diagnostic Medical Device (for USA)



Lot number



Manufacturer



Medical device (for EU/UK)



Reference number



Recyclable Polypropylene Plastic



Serial number



Temperature limitations



Use - by date



UK Conformity Assessed



Qurasense, Inc.
3517 Edison Way, Ste. D
Menlo Park, CA 94025
United States of America



Speculo Limited
10/1, Sir M.A. Refalo Avenue
Balzan BZN 1600
Malta



Speculo Ltd
107-111 Fleet Street
Ludgate House
London, EC4A 2AB
United Kingdom



QHS.1.2

2026.01.01